In-situ hardening alloplastic bone graft system

Composition

easy-graft CRYSTAL contains:

- Biphasic calcium phosphate (60 % hydroxyapatite, 40 % beta-tricalcium phosphate) with poly(lactide-co-glycolide) (PLGA)
- BioLinker (N-Methyl-2-pyrrolidone-solution)

Description

easy-graft CRYSTAL is an alloplastic bone defect filler. It consists of two components: granules (syringe) and BioLinker (ampoule). After mixing, *easy-graft* CRYSTAL is putty-like and can be applied directly from the syringe into the bone defect. In contact with body fluids, *easy-graft* CRYSTAL hardens within minutes to form a solid but porous implant.

easy-graft CRYSTAL is biocompatible and osteo-regenerative. For biphasic calcium phosphate and PLGA in adequate form, no inflammatory or immunoreactions are known.

easy-graft CRYSTAL forms a stable and porous body within the defect. The granules already feature a high porosity. After placing the material in the defect, the pore volume allows the uptake of blood. With the integration of blood, important factors for tissue regeneration are incorporated. In a second step, autogenous tissue can grow into the pore structure.

During the initial phase of degradation, *easy-graft* CRYSTAL may swell by taking up body fluids, thus supporting a close contact to the surrounding bone tissue. When applied in larger defects, this might result in slight sensation of pressure by the patient. *easy-graft* CRYSTAL contains no animal or human derived substances. Biphasic calcium phosphate and PLGA are derived from tully synthetic raw materials.

Indications

easy-graft CRYSTAL should be used for the filling of mostly unloaded and preferably multi-walled dental or maxillo-facial bone defects. The implantation site should be free of infections and soft or granulation tissue. Indications may include:

- Defects after removal of bone cysts
- Periodontal defects
- Augmentation of alveolar crest (e.g. Guided Bone Regeneration, GBR)
- Defects after root end resection
- Extraction defects
- Defects after surgical removal of retained teeth
- Sinus floor elevations
- Defects after removal of autologous bone

Application

Aseptic working methods are essential during application. *easy-graft* CRYSTAL is provided in the particle sizes of 450 – 630 µm for smaller defects and 450 – 1000 µm for larger defects. Dental membranes may be used in combination with *easy-graft* CRYSTAL at the discretion of the practitioner.

Limitation of use / Precautions

Contraindications
easy-graft CRYSTAL must not be used during pregnancy. *easy-graft* CRYSTAL should not be applied in case of the presence of any diseases or therapies which are detrimental to the healing of bone. No allergic reactions against *easy-graft* CRYSTAL or its components are known. However, if an allergic reaction is suspected, the product should not be applied in the affected patient.

Precautions

In order to facilitate the formation of new bone to a maximum extent, *easy-graft* CRYSTAL should only be implanted in close contact to well vascularized surrounding bony tissue (refreshment of bleeding might be necessary). Care should be taken that *easy-graft* CRYSTAL granules are not introduced into surrounding soft tissue, because the granules will only partially resorb.

Periodontology

Before filling periodontal defects, a sufficient pre-treatment is necessary. This should include sufficient plaque control and a successful local treatment of the periodontal lesion (e.g. root planning, debridement).

Possible adverse effects

Allergic or inflammatory reactions to the implant material or its decomposition products have not been recorded.

Any serious incident that has occurred in relation to *easy-graft* CRYSTAL should be reported to the manufacturer and the competent authority of the country where the dental professional and/or patient is established.

Interaction with other substances

Not known.

Warnings

easy-graft CRYSTAL is restricted to dental and maxillo-facial applications and is to be used only by the order of a licensed physician or dentist.

easy-graft CRYSTAL must not be applied to acute infected defects.

Do not place *easy-graft* CRYSTAL in direct contact with nerve tissue.

The product has not been clinically tested in children.

Lactation: It is not known whether metabolites of *easy-graft* CRYSTAL pass into breast milk.

BioLinker contains N-Methyl-2-pyrrolidone. N-Methyl-2-pyrrolidone is irritant to eyes, respiratory system and skin. Wear gloves and protections goggles when working with BioLinker. Change gloves after contact with BioLinker.

After eye contact: In case of contact, immediately flush eyes with copious amounts of water for at least 10 minutes and consult a doctor.

Do not use if package is opened or damaged, or if the expiration date has been exceeded.

easy-graft CRYSTAL cannot be reused. Discard any unused material. Sterility and function are no longer guaranteed in the event of re-use.

Properties

easy-graft CRYSTAL consists of biphasic calcium phosphate, PLGA and BioLinker. Resorption proceeds via three steps:
1) The BioLinker is metabolized and excreted within days mainly through the urine.
2) The absorbable polymer (PLGA) is cleaved by hydrolysis. The degradation products are excreted.
3) The biphasic calcium phosphate is partially resorbable by cellular mechanisms and direct dissolution. A part of the biphasic calcium phosphate remains embedded in autogenous tissue for prolonged periods of time in order to preserve the volume.

easy-graft CRYSTAL supports bone regeneration and bone formation. The application of the material does not guarantee a treatment success. The success of the therapy depends on many factors such as surgical technique and habits, age and the regeneration potential of the patient.

Miscellaneous

easy-graft CRYSTAL is gamma sterilized and intended for single application. Unused material can be disposed in the domestic waste.

Storage instructions

easy-graft CRYSTAL must be stored in its original packaging. Store between 5 °C to 25 °C. Keep away from sunlight.

Date of issue

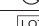
2016-07-01

Manufacturer

Degradable Solutions AG*, Wagistrasse 23, 8952 Schlieren / Zurich, Switzerland

*A Sunstar group company

www.guidor.com

	CE Marking - Notified body: DQS Medizinprodukte GmbH	REF	C15-002 C15-012 C15-072 C15-005 C15-015 C15-075
	Catalogue number		Do not re-use
	Batch code		Use-by date
	Consult instructions for use		Sterilized using irradiation
	Temperature limit		Keep away from sunlight
	Do not use if package is damaged		Manufacturer

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Degradable Solutions AG*, Wagistrasse 23, CH-8952 Schlieren / Zürich, Suiza